ENGLISH

SYMBOLS

MD Medical Device

INTENDED USE

The device is intended to restrict motion of the wrist and/or thumb

Indications for use

For conditions that may benefit from restricting motion of the wrist, such as

- Mild to moderate sprains and strains
 - Tendinitis
 - Post-cast healing
 - Other selected soft tissue injuries.
- No known contraindications.

Warnings and Cautions:

- Inspect the skin under the device's contact area before and after each use.
- The device should be used directly on intact skin and never over an open wound.
- Care should be taken not to overtighten the device.
- Patients with already compromised blood circulation in their extremities, like those with diabetes or peripheral vascular disease, need to be extra careful and are advised to consult with a healthcare professional before using the device. If you experience pain, swelling, sensation changes, or if your extremity shows signs of insufficient blood flow (turns blue, white, or cold) while using this device, discontinue the use immediately and consult with a healthcare professional.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference. Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

- The patient should stop using the device and contact a healthcare professional:
 - If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
 - If any pain, skin irritation, or unusual reaction occurs with the use of the device.
- The device is for single patient – multiple use.

USAGE

Device Application

Positioning of the brace:

Align the palmer stay to the palm of the hand (Fig. 1).

Putting on the brace:

- Undo all contact closure straps (Fig. 2).
- Slide hand into the brace and place thumb into thumb hole. Ensure stockinette is positioned on top of the hand (Fig. 3).
- Fasten all contact closure straps and secure to a snug fit (Fig. 4, 5). Adjustments may be necessary as hand, wrist, and thumb swelling patterns change.
- Straps may be cut as desired (Fig. 6).

Notes:Adjustments on the aluminum palmer stay can be performed by a healthcare professional.

Cleaning and care

- Remove aluminum stays before washing.
- Hand-wash using mild detergent and rinse thoroughly.
- Air dry, do not expose to direct heat.
- Do not squeeze or wring out the device.

Note:Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.

Note:Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Osour does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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