

Instructions for Use

COMFORT SCS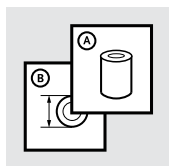


		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	10
ES	Instrucciones para el uso	13
IT	Istruzioni per l'uso	16
NO	Bruksanvisning	19
DA	Brugsanvisning	22
SV	Bruksanvisning	25
EL	Οδηγίες Χρήσης	28
FI	Käyttöohjeet	31
NL	Gebruiksaanwijzing	34
PT	Instruções de Utilização	37
PL	Instrukcja użytkowania	40
CS	Návod k použití	43
TR	Kullanım Talimatları	46
RU	Инструкция по использованию	49
JA	取扱説明書	52
ZH	中文说明书	55
KO	사용 설명서	58

①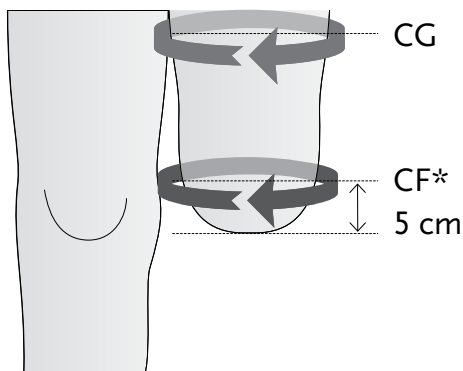

②

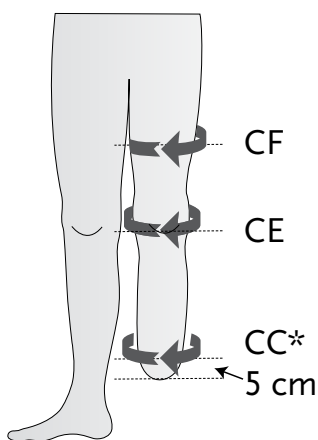DESCRIPTION

The device is a sock made of compressive knitted fabrics which can be used for compression therapy.

INTENDED USE

The device is intended to provide compression therapy.

Suitability of the device for the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

Indications for use

Lower limb amputation and/or congenital deficiency.

- Assisting in edema reduction
- Supporting prevention of edema
- Reducing wound healing time
- The shorter wound healing time reduces the time to prosthetic fitting

Contraindications

Do not use if a clinical examination shows that no compression must be applied, especially due to:

- Open wounds
- Arterial occlusive disease
- Decompensated heart failure
- Congestion dermatitis
- Acute thrombophlebitis
- Unreduced edema

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

Caution: Do not wear the device with a prosthesis.

DEVICE SELECTION

Caution: The wrong size can lead to discomfort, skin irritations, or a less reliable function.

Size	CF*	CG
	Circumference in cm	
I	30 - 32	42 - 45
II	32 - 35	45 - 49
III	35 - 38	49 - 53
IV	38 - 41	53 - 57
V	41 - 44	57 - 61
VI	44 - 47	61 - 65

Length of Thigh

Extra long 33 - 37 cm Long 28 - 32 cm

Middle 23 - 27 cm Short 18 - 22 cm

Size	CC*	CE	CF
	Circumference in cm		
0	26 - 28	28 - 30	38 - 40
I	28 - 30	30 - 32	40 - 42
II	30 - 33	32 - 35	42 - 45
III	33 - 36	35 - 38	45 - 48
IV	36 - 39	38 - 41	48 - 51
V	39 - 42	41 - 44	51 - 54

Length of Lower Leg

Long 42 - 48 cm Middle 35 - 41 cm

Short 28 - 34 cm

USAGE

Caution: Open wounds or damaged skin should be covered with a bandage or other appropriate covering to prevent direct contact between the wound and device.

Donning

- Turn the device inside out until the inner surface is fully exposed almost to the tip.
- Stretch the device with both hands and slip on the residual limb.
- Slide the device up the residual limb, starting at the bottom and carefully spreading the material over the limb.
- Make sure that the device does not have wrinkles.

Caution: Take care not to damage the device with fingernails or jewelry.

Caution: The device should not be cut or shortened.

Cleaning and care

The device must be washed carefully before first use.

Daily cleaning of the residual limb and the device is essential. Use of a mild, pH balanced, 100% fragrance- and dye-free liquid soap is recommended. After washing rinse the device several times.

The device can also be machine washed (40 °C) with a mild detergent and at low spin speed. Fabric softeners, bleaches, and other products/ cleaning solutions may result in device damage and should not be used. Hang device up to dry or spin dry immediately using the delicate wash setting.

Do not leave to dry on heaters or in direct sunlight.

Do not iron. Do not dry clean.

Caution: Skin health should be monitored daily. In case of abnormal symptoms, discontinue use of the device and contact the healthcare professional immediately.

Caution: Be aware that common household or bath products may cause or contribute to skin irritation, e.g., soaps, deodorants, perfumes, abrasive cleaners, aerosol- or alcohol sprays.

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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