
Instructions for Use




---

VACU VALVE

1810723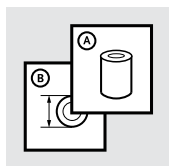

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	6
FR	Notice d'utilisation	8
ES	Instrucciones para el uso	10
IT	Istruzioni per l'uso	12
NL	Gebruiksaanwijzing	14
PT	Instruções de Utilização	16
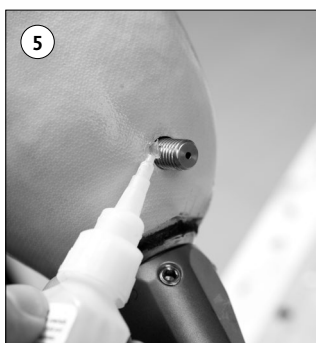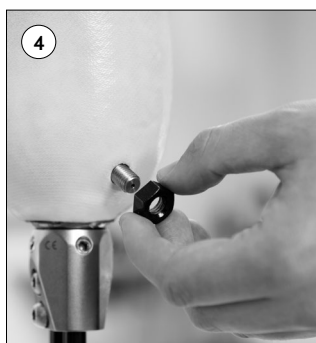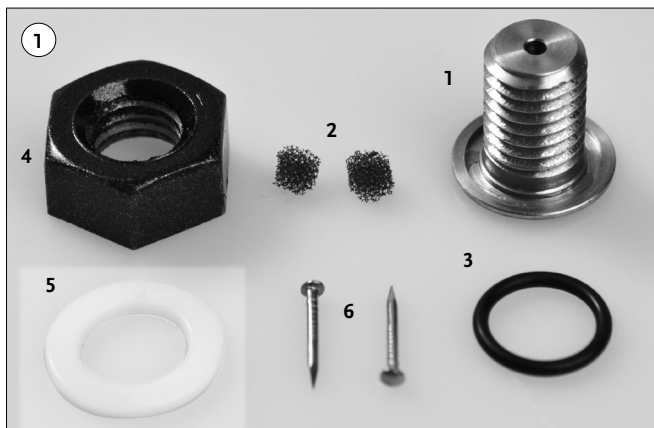
### DESCRIPTION

The Vacu valve is an auto expulsion valve for use in a vacuum socket system.

### Components (Fig. 1)

1. Air Expulsion Valve
2. Filter
3. Gasket
4. Plastic Nut
5. Dummy Disc
6. Nails

### INTENDED USE

The device is intended to connect and release a prosthetic system that replaces a missing lower limb.

The device must be fitted and adjusted by a healthcare professional. Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

It is intended to be used with a cushion liner and a sleeve.

### Indications For Use and Target Patient Population

- Lower limb amputation and/or congenital deficiency
- No known contraindications

The device is for high impact use, e.g., walking and occasional running.

### GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

**Warning:** If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

### ASSEMBLY INSTRUCTIONS

The device can be used for both thermoplastic and laminated socket, with wall thickness between 2 and 7 mm.

1. Position the dummy disc supplied in a suitable place on the distal end of the plaster model and secure it with the nails supplied (**Fig. 2**).
2. Fabricate the socket as usual.
3. Drill a 10 mm hole in the middle of the dummy and deburr the hole (**Fig. 3**).
4. Pass the valve from the inside of the socket through the hole and then screw the valve finger-tight into the nut supplied (**Fig. 4**).
5. As a precaution with interim and definitive sockets you can also seal the joint between the edge of the hole and the valve with silicone adhesive (**Fig. 5**).

## **USAGE**

### **Cleaning and care**

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

### **Environmental Conditions**

The device is Weatherproof.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e. g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

## **MAINTENANCE**

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity. If the filter becomes dirty after wearing for a certain period, you can e.g., remove the filter with small tweezers and replace it with the replacement valve supplied.

## **REPORT OF SERIOUS INCIDENT**

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

## **DISPOSAL**

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## **LIABILITY**

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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